


Alcian Blue (pH 0.5) Stain Kit

Description: The Alcian Blue (pH 0.5) Stain Kit is intended for use in the histological visualization of strongly sulfated mucosubstances.

Strongly Sulfated Mucosubstances: Blue
Nuclei: Red
Background: Pink

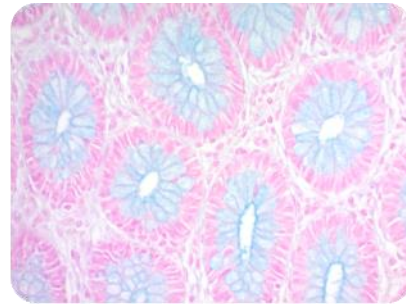

Uses/Limitations: Not to be taken internally.
For In-Vitro Diagnostic use only.
Histological applications.
Do not use past expiration date.
Use caution when handling reagents.
Non-Sterile.

Control Tissue: Small Intestine
Appendix
Colon

Availability/Contents:


<u>Item #</u>	<u>Kit Contents</u>	<u>Volume</u>	<u>Storage</u>
ANB250	Alcian Blue Solution (pH 0.5)	250 ml	18-25°C
AAG500	Acetic Acid Solution	500 ml	18-25°C
NFS250	Nuclear Fast Red (Enhanced Stability)	250 ml	18-25°C

Precautions: Avoid contact with skin and eyes.
Harmful if swallowed.
Follow all Federal, State, and local regulations regarding disposal.
Use in chemical fume hood whenever possible.

Procedure (Standard):

1. Deparaffinize sections if necessary and hydrate to distilled water.
2. Incubate slide in Acetic Acid Solution for 3 minutes.
3. Stain tissue section with Alcian Blue Solution (pH 0.5) solution for 30 minutes at room temperature or 15 minutes at 37° C.
4. If desired, rinse slide briefly in Acetic Acid solution to remove excess Alcian Blue.
5. Rinse for 2 minutes in running tap water followed by 2 changes of distilled water.
6. Stain tissue section with Nuclear Fast Red (Enhanced Stability) for 5 minutes.
7. Rinse for 2 minutes in running tap water followed by 2 changes of distilled water.
8. Dehydrate through graded alcohols.
9. Clear, and mount in synthetic resin.

Storage: 18° C  25° C



ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
U.S.A.

CE 

EC REP

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem, The Netherlands

Instructions For Use

AFS-1-IFU


Rev. Date: Dec. 8, 2015

Revision: 2

Page 2 of 2

P.O. Box 3286 - Logan, Utah 84323, U.S.A. - Tel. (800) 729-8350 – Tel. (435) 755-9848 - Fax (435) 755-0015 - www.scytek.com**References:**

1. Lillie, R.D. 1977, H.J. Conn's Biological Stains, 9th Edition. Williams & Wilkins, Baltimore. Pages 452-455.
2. Sheenan, D.C., Hrapchak, B.B. Theory and Practice of Histotechnology, 2nd Edition. Battelle Press, Columbus, OH. Pages 172-173.
3. Churukian, C.J., 1989, Manual of Special Stains Laboratory, 4th Edition. University of Rochester, Rochester, New York. Pages 55-56.
4. Carson, F.L., 1996, Histotechnology; A Self-Instructional Text, 2nd Edition. ASCP Press, Chicago, IL. Pages 117-121.

Storage:
18° C  25°
C ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
U.S.A.CE 

EC REP

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem, The Netherlands